# ACUMED®

## ACUMED SCAPHOID TARGETING GUIDE

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## ACUMED SCAPHOID TARGETING GUIDE

### FOR THE PERSONAL ATTENTION OF THE OPERATING SURGEON

**DESCRIPTION:** The Acumed Scaphoid Targeting Guide is designed for use with Acutrak screws in percutaneous fixation of non-displaced fractures of the scaphoid. Proper care and inspection after each use is critical.

**INFORMATION FOR USE:** The Acumed Scaphoid Targeting Guide is reusable; however, its components have a limited life span. Prior to and after each use, the components must be inspected where applicable for wear, damage, proper cleaning, and integrity of the connecting mechanisms.

**WARNINGS:** For safe and effective use of the Acumed Scaphoid Targeting Guide, the surgeon must be familiar with the device, the method of application, and the recommended surgical technique. Breakage or damage can occur when the device is subjected to excessive loads or improper technique. The patient must be cautioned, preferably in writing, as to the risks associated with the device.

**CLEANING:** The Acumed Scaphoid Targeting Guide must be thoroughly cleaned before reuse. Decontamination of the device should occur immediately after completion of the surgical procedure. Excess blood or debris should be wiped off to prevent it from drying onto the surface. Using an enzymatic cleaning product such as Enzol, prepare the cleaning solution according to manufacturer's instructions.

Avoiding prolonged soaking, submerge the device in the enzymatic cleaning solution for 5 minutes. After 5 minutes, use a soft bristle brush to remove any detergent residue. Instruments with lumens also should be held under water (larger side up) ensuring the lumen is flushed with water. Place the device in an ultrasonic cleaner filled with deionized water, ensuring that there is no air trapped in any of the crevices by gently agitating the device. Sonicate the device for 5 minutes. Remove the device from the ultrasonic cleaner and rinse in water by submerging it completely. Gently wipe the device with a clean, dry cloth, then air dry.

**STERILITY:** This product is provided non-sterile. Sterilization may be performed using one of the following methods.

#### Sterilization Methods

|                                |   |
|--------------------------------|---|
| Gravity Displacement Autoclave | <b>Full Cycle Time:</b> 270° F (132° C) for 30 minutes<br><b>Dry Time:</b> 45 minutes |
| Pre-Vacuum Autoclave           | <b>Full Cycle Time:</b> 270° F (132° C) for 4 minutes<br><b>Dry Time:</b> 70 minutes  |

- Consult your equipment manufacturer's written instructions for specific sterilizer and load configuration instructions.
- Follow current AORN "Recommended Practices for Sterilization in Perioperative Practice Settings" and ANSI/AAMI ST79: 2006 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- Flash sterilization is not recommended, but if used, should only be performed according to requirements of ANSI/AAMI ST79: 2006 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.

**STORAGE INSTRUCTIONS:** Store in a cool dry place and keep away from direct sunlight. Prior to use, inspect product package for signs of tampering, or water contamination.

**CAUTION:** For Professional Use Only.

**CAUTION:** Federal Law (USA) restricts this product sale by or on the order of a physician or hospital.

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